# ENGLISH

**The Aeroneb Solo Adapter** is an accessory specific to the Aeroneb Solo nebulizer. It facilitates intermittent and continuous nebulization and optional supply of supplemental oxygen to pediatric and adult patients via mouthpiece. The device can alternatively be used with a facemask, which is not provided.

It is a single patient use device which is qualified for 20 intermittent use treatments (at a rate of four 3ml doses per day over 5 days) or 3 hours of continuous use.

For continuous use, the Aeroneb Solo Adapter can be used in conjunction with the Aeroneb Solo Continuous Nebulization Tube Set. See Aeroneb Solo Instruction Manual.

Optimal aerosol delivery is achieved with valved mouthpiece or valved aerosol facemask with low/no oxygen flow.

## Aeroneb Solo Adapter Kits

AG-AS7500-EU: Aeroneb Solo Adapter Starter Kit (10 pack): Aeroneb Solo, Adapter, oxygen tubing

AG-AS7000-EU: Aeroneb Solo Adapter (10 pack): Adapter, oxygen tubing

### INSTRUCTIONS FOR USE - Refer To Figures On Page 1

Inspect for device integrity and correct valve placement prior to use.

- 1** Insert Aeroneb Solo firmly into Adapter in orientation shown.
- 2** If supplemental oxygen is required, firmly attach oxygen tubing to Adapter.  
**Note:** Oxygen flow rate should be set between 1-6 LPM.
- 3** If a facemask is required, remove mouthpiece and attach facemask to Adapter.  
**Note:** When using an open facemask, a minimum oxygen flow of 1 LPM is required.
- 4** Add prescribed medication to nebulizer as instructed in Aeroneb Solo System Instruction Manual.
- 5** Connect cable to Aeroneb Solo.  
**Note:** Refer to Aeroneb Solo Instruction Manual for intermittent and continuous nebulization set-up and operation instructions.
- 6** Power on and observe Adapter to confirm aerosol is visible.
- 7** Introduce Adapter to patient and observe aerosol flow to ensure correct operation.
- 8** Remove excess rainout from Adapter periodically (hourly with continuous nebulization). To ensure optimum performance of Adapter, remove any residue by rinsing through with sterile water, shake off excess and allow to air dry.

## WARNINGS

### Performance

Do not use with a closed facemask.

When using with an open facemask, always use supplemental oxygen flow of 1-6 LPM.

Performance of Adapter may vary depending upon the type of drug and Adapter configuration used.

Do not exceed recommended oxygen flow for system.

Visually check Adapter post-rinsing to ensure that valves have not become dislodged.

### General

To be used by trained personnel only.

Use only with recommended components.

Ensure tubing is safely orientated to prevent strangulation hazard.

Ensure oxygen connection port or tubing is not occluded.

Do not use Adapter without a mouthpiece or facemask.

Do not autoclave any component of the kit.

Do not cover Adapter valves during use.

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© 2014 Aerogen Ltd.
Aerogen Ltd.